

Lamdown[®]

UNPOWERED AIR PURIFYING PARTICULATE RESPIRATOR

KN95 Protection Level

Instruction

1. The user shall receive necessary training before use and wear it in the correct way.
2. This product can only be used in a ventilated place with sufficient oxygen.
3. If the mask is blocked and has difficulty breathing or is damaged, please discard the mask or replace it.
4. If breathing is difficult or unpleasant while wearing a mask, leave the work-place immediately and breathe fresh air.

Important Information

1. It is not recommended to use this product when oxygen content is low, air pollution is serious or when sleeping.
2. Use with caution on patients with asthma and other respiratory diseases.
3. Please use the mask correctly and do not abuse it
4. The storage temperature is -28°C-38°C, and the relative humidity is lower than 80%.

Wearing Instructions


1. Open the mask. Hold the ear strips while the nose clip is facing the outside.
2. Position the bottom of the mask under your chin and the nose clip against bridge of your nose.
3. Pull the ear straps behind the ear.
4. Using both hands, mould metal nose piece comfortably to shape of nose.

Pinch the nose piece using one hand may result in less effective respirator performance.



Filtering effect $\geq 95\%$

- Three-dimensional
- Non-woven fabric
- Bridge bar adjustable
- Fusion-jet filter material

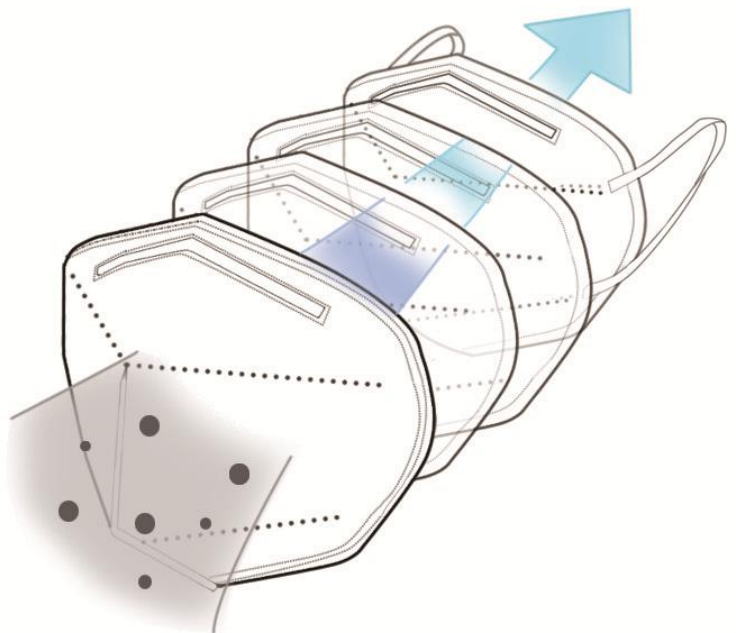

FDA

CE



Name : UNPOWERED AIR PURIFYING PARTICULATE RESPIRATOR

Model : SD-KN95

Category: disposable, no breathing valve
Standard: GB2626-2006

Shelf life: 3 years

Production date: see the seal

Tel: 0746-2999789

Manufacturer: LANSHAN SHENDUN Technology Co., Ltd

Address: No.5 leather industrial park , Lanshan county, Hunan province

